IZYTRONIQ

Release Notes

3-447-083-15
7/4.22

D

Version 04.07 (QF1)

Neuerungen

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Version 04.07 (QF1)

News

- SECUTEST ST BASE(10) / PRO and SECULIFE ST BASE(25) / PRO with firmware version 03.03.01 are supported.

Bug Fixes

- Device drivers are correctly installed.
- Interface software IZYIONIQ starts correctly. (Missing rights have been implemented.)

Activities

- Business
 - Carry out a manual update of the client installation. You find the update file at www.izytron.com. Alternatively, the update is performed with a delay via the update server. (Observe the following: The update server might be deactivated temporarily.)

Note!

Be sure to back up your database before starting the update. The database structure is automatically updated during update.

- Enterprise
 - Carry out an update of the backend (server). You have to carry out the update manually. You find the update file at www.izytron.com.
 - Carry out an update of the client installation. You have to carry out the update manually. You find the update file at www.izytron.com.

Note!

Be sure to back up your database before starting the update. The database structure is automatically updated during update.

- Cloud
 - We will update the cloud backend. You will be given notice to this end. Subsequently, you have to update your client installation. If you have activated the automatic update function via the Internet, you will be prompted to carry out an update. If it is deactivated, you have to carry out the update of the client installation manually. Download the current software via your cloud account (at www.gossenmetrawatt.cloud) for this purpose and for new installations.

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GB

Version 04.07**Note!**

Previous versions of IZYTRONIQ are now available for download at www.izytron.com/downloads.php.

News

- Password forgotten: Passwords can be reset provided that a valid e-mail address has been saved to the program and a connection with the Internet exists.
- Revised user menu with display of user name, access to personal settings and logout.
- Simplified login: The user name of the most recently logged-in user is displayed again in the login dialog.
- New, more clear-cut home screen. The dashboard must now be accessed and is displayed separately.
- Access to tabs “Tests” and “Measurement Series” can be restricted (via the rights and roles management function).
- Location information is additionally shown in the first tab “Master Data”.
- Supplementary fields: 3 new fields in the first tab “Master Data” which can be labeled individually and filled with free text. The fields can be searched and included in reports. (Configuration in “Global Settings”).
- Stationary objects can be moved within the tree structure (in compliance with the predefined object dependency).
- In the report for portable objects the visual inspection can be displayed on a reduced scale (as 1 step).
- Support of METRAHIT measurement series (multimeter live data).
- In the conflict manager all conflicts can be solved at once now.

Improvements

- A progress bar is displayed when a big number of objects is deleted.
- Improved display of the selected dashboard filters.
- Export to the instrument is faster now.
- Deletion of tree structure selection (quick links) has been improved.
- Display of the currently occupied floating licenses has been optimized and extended to include ULTIMATE licenses.
- Size of column editor has been adapted.
- Template for submitting error messages per e-mail has been optimized.

Bug Fixes

- Offline mode of Cloud variant works faultlessly.
- Floating licenses are counted correctly when a user logs out or the program crashes.
- Changes in many objects can be saved.
- Stationary objects can be created in an error-free fashion.
- Stationary objects are filtered correctly.
- New fields in the extended master data are stored correctly.
- The field “next test” is updated in the case of changes.
- Parameter “Conductor cross-section” can be defined correctly. Increment = 0.01mm².
- After deleting objects, the number of listed objects is updated accordingly in the display.
- Faster display of the e-tree of stationary objects when the latter contains many RCDs.
- If a faulty entry is made when creating the first user, the incorrect field is highlighted.
- If no test name has been entered, an error icon is shown in the respective field to make reference to the error.

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GB

- IZY export functions flawlessly.
- Upon importing a backup, the correct text appears in the completion message.
- Newly created customers are immediately listed in the dropdown menu upon import (no program restart necessary any more).
- The “Close” button is deactivated during import until import is completed. This prevents accidental termination of the import process with resulting data loss.
- Import of SECUTEST file (*.secu) supplies correct information about test equipment.
- Object lists can be printed even if columns are hidden.
- Only those object properties which are available for the object are taken into account for printout.
- The date of the next test is now correctly scheduled in the report.
- Before a report template is opened in Microsoft® Word for editing, it is checked as to whether it has been saved. If this is not the case, a prompt appears asking you to save the template. (A non-saved template would be opened as “read only”).
- Missing translations in French templates completed.
- In the remote test it is possible to switch between the results and the sequence overview.
- In the case of manual entry in remote sequences, values can be presented with places behind the decimal point.
- When creating a sequence for PROFITEST instruments, the sequence step “user-analyzed measurement” can be included again.
- Sequence operations with PROFITEST PRIME are working again.
- Performance problems with remote measurements with SECUTEST/SECULIFE ST instruments have been remedied.
- The sequence editor now works for SECUTEST/SECULIFE ST instruments with firmware version 2.1.1.
- The sequence editor now works for PROFITEST instruments.

Activities

- Business
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay. Backup your database before the update. The database structure is updated automatically with the update.
- Enterprise
 - Perform an update of the back end (server). The update has to be carried out manually. You find the update file on www.izytron.com. Backup your database before the update. The database structure is updated automatically with the update.
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay.
- Cloud
 - We will update the cloud back end. You will receive a notification for this and must then update your client installation. If the automatic updates via Internet are activated, you will receive a prompt to update. If they are deactivated, the client installation update has to be carried out manually. To do so and for new installations, download the newest file from your cloud account (www.gossenmetrawatt.cloud).

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Version 04.06 (SP2) (QF1)

Improvements

- Import: If the recycle bin is filled with data, this is now indicated
- “Active”/“Inactive” device status:
 - If there is a new test for an inactive device during a re-import, the device is set to “active”. In all other cases the device remains “inactive”.
 - Can be changed for several devices using the multi-assistent.

Bug Fixes

- Cloud: If a role is created, the login permissions are set correctly.

Activities

- Business
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay. Backup your database before the update. The database structure is updated automatically with the update.
- Enterprise
 - Perform an update of the back end (server). The update has to be carried out manually. You find the update file on www.izytron.com. Backup your database before the update. The database structure is updated automatically with the update.
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay.
- Cloud
 - We will update the cloud back end. You will receive a notification for this and must then update your client installation. If the automatic updates via Internet are activated, you will receive a prompt to update. If they are deactivated, the client installation update has to be carried out manually. To do so and for new installations, download the newest file from your cloud account (www.gmc-instruments.cloud).

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GB

Version 04.06 (SP2)**Bug Fixes**

- Devices are loaded correctly when called through the dashboard.
- Data import: Object parameters are refreshed when displayed.
- Import preview: Faulty feature to change a customer was removed.
- Standard customer: Can only be chosen during CSV import, the faulty display in other dialogs has been removed.
- All values such as nominal current and cross section now have units.
- Protocols: Display (printing) works again.
- PROFITEST distributor report: All devices used for testing are now listed.
- Emptying the recycle bin: Empties entire bin again. (Each user with rights to empty the bin can now irrevocably delete all users that are in the bin.)
- Reset to default settings: Is carried out without errors now.
- Global settings: Entries are displayed correctly now.
- Language: If the system language is not available, English will be used as program language. (Crash fix)
- Enterprise: Objects of several users can now be deleted simultaneously.
- Ultimate: All conflicts of a database on hold can be resolved now.
- Ultimate/Offline Cloud:
 - Offline databases can be synchronized again.
 - Objects are deleted across offline clients now.
- Cloud:
 - If there is a problem while switching to the offline mode, the client will automatically get back to online mode.
 - IZY file import works again.

Activities

- Business
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay. Backup your database before the update. The database structure is updated automatically with the update.
- Enterprise
 - Perform an update of the back end (server). The update has to be carried out manually. You find the update file on www.izytron.com. Backup your database before the update. The database structure is updated automatically with the update.
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay.
- Cloud
 - We will update the cloud back end. You will receive a notification for this and must then update your client installation. If the automatic updates via Internet are activated, you will receive a prompt to update. If they are deactivated, the client installation update has to be carried out manually. To do so and for new installations, download the newest file from your cloud account (www.gmc-instruments.cloud).

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Version 04.06 (SP1)

News

- Users with the role „Tester“ are now enabled to resolve conflicts.

Bug fixes

- The location tree is displayed in the record again.
- Import of PROFITEST devices with empty IDs is working.
- The IDs are displayed in the PROFITEST distributor report.
- The nominal current value of the overcurrent protection device is displayed in the PROFITEST distributor report.
- All master data fields are adopted during import from SECUTEST-S4 devices.
- Cloud version: The recycle bin can be opened without any difficulty.

Activities

- Business
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com. Alternatively, the update will be carried out via the update server with a delay. The database structure is updated automatically with the update.
- Enterprise
 - Perform an update of the back-end (server). The update has to be carried out manually. You find the update file on www.izytron.com. The database structure is updated automatically with the update.
 - Perform an update of the client installation. The update has to be carried out manually. You find the update file on www.izytron.com.
- Cloud
 - Perform an update of the client installation. This can be done via the update server. For new installations, download the newest file from your cloud account (www.gmc-instruments.cloud).
- All versions
 - If you have modified the tester role, check as to whether the tester role still corresponds to your settings after the update.
 - Device status “Active” and “Inactive”: When importing from the test instrument into the software, the “Active” field is always set to “YES“. Inactive devices thus become active again. If you use the status function, you have to filter these devices when exporting from the software back into the test instrument in order to prevent an unwanted activation.

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GB

Version 04.06**News**

- Offline functionality for the cloud version
- Display of back-end version in the client
- IZY file export with attachments
- „Active“ and „Inactive“ device status
- PROFITEST distribution log: Tabular list of measurement results for a better overview

Improvements

- E-mail will become mandatory for all users to allow for the development of future features
- Users will be required to perform an update of the database to the currently used version
- Progress bar has been added for save process
- Progress bar for deletion of large amounts of objects added
- Duration of exports to the device has been reduced
- Export (csv) from the list of objects improved
- File import: „RESOLVE BY ID“ has been enabled
- Report containers can be canceled.
- Filter "Empty field" added
- Filtering for column „Deadline status“ possible
- Filter function for object type added
- Column „Standard“ in the catalog is masked as it cannot be modified
- Polarity „L/N“ and „NL“ replaced by „Normal“ and „Reversed“
- Comments added to „Push/Print“
- Comments added to „Manual test“

Bug Fixes

- General improvements of the overall system
- Ultimate: Error when canceling a database put on hold has been solved
- System requirements updated for the client installation on multi-core systems with low core pulsing
- Error during report generation solved
- Multiple entry of an object due to repeated changing of location eliminated
- Changes effected via the MULTI-assistant for import preview are implemented
- Evaluation of input values during manual entry of measured values for remote measurements has been corrected
- New clients are listed in drop-down menu without restart
- Close button is not shown for imports until transaction has been completed
- Print filter list: Percentage calculation corrected
- Sorting comment field: Drop-down menu opens
- Labeling in the global settings „Show test deadlines“ adapted to „Show calibration deadline“
- The functions „Repeat“ and „Skip“ have been eliminated for a remote test instruction

Activities

- Business
 - Perform an update of the client installation. This can be done via the update server.
 - The database structure is updated.
- Enterprise
 - Perform an update of the back-end (server). The update has to be carried out manually.
 - Perform an update of the client installation. The update has to be carried out manually.

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GB

- Cloud
 - Perform an update of the client installation. This can be done via the update server.

Version 04.05 (SP1) (QF4)

News

- Communication with SECUTEST S4 and the current firmware 3.1.0 provided

Activities

- An update of the client installation must be performed
- The server installation remains unaffected

Version 04.05 (SP1) (QF3)

Bug fixes

- Filter speed accelerated in the business version
- Improved setting of default values (default parameters) when importing attachments

Activities

- An update of the client installation must be performed

Version 04.05 (SP1) (QF2)

Bug Fixes

- Red marked ID and name and error message "Object reference was not created on object instance" fixed
- Collective report generation for stationary objects possible again
- Object with same key has already been added has been solved
- Selection in recycle bin marking wrong record has been fixed
- Deadlock when loading the tree has been fixed
- Crash when opening the recycle bin when user no longer exists solved [Cloud]

Activities

- An update of the client installation and the server installation must be performed

Version 04.05 (SP1) (QF1)

News

- Reciprocal tree marking was deactivated by default. This function is only recommended for small amounts of data and can be activated in the personal settings
- Extended search functions "Selection list" are deactivated by default. This function is only recommended for small amounts of data and can be activated in the personal settings
 - This limits the filters that are active by default
- Communication with PROFTEST improved.
- Communication with SECUTEST improved.

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GB

Improvements

- Further selection options integrated in the new filters
- When creating new objects, the ID can be used as a description.

Bug Fixes

- Improved role and rights management in IZYTRONIQ
- A progress bar is displayed when creating multiple logs
- Filtering of the date range during log creation has been improved
- Timeouts during communication with the SECUTEST fixed
- Dialog "New element" adapted to the new functions
- Incorrect multiple location assignment has been removed
- Measured value table in Push-Print mode has been revised
- Updating the table after location allocation has been revised
- No empty database is played into the SECUTEST